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CONTENTS

Safety precautions	31
Introduction	32
Precautions before use	33
Functions	33
Usage	34
Practical considerations when taking a temepature	34
How to take your dog's temperature	35
Points of insight	36
Description	37
Setting and function	37
Instructions for calibration	39
Changing the batteries	39
Technical characteristics	40
Troubleshooting	41
Warranty	43

The manufacturer reserves the right to alter the specification of the product without prior notification.

SAFETY PRECAUTIONS

- Follow the maintenance advice stipulated in this instruction manual.
- NOT FOR USE ON HUMANS!
- This device must only be used in an ambient temperature range of between 10°C and 40°C.
- This device must always be kept in a clean, dry area.
- Do not expose this thermometer to electric shocks.
- Do not expose this thermometer to extreme temperature conditions of > 50°C or < 0°C.
- Do not use this device in relative humidity higher than 85%.
- The protective glass over the lens is the most fragile part of the thermometer.
- Do not touch the glass of the infrared lens with your fingers.
- Clean the glass with a cotton bud lightly moistened with 70% alcohol.
- Do not expose the thermometer to sunlight or to water.
- Never drop the device.
- Do not attempt to repair this device yourself. If you have any problem, please, contact authorised service department.

EN

To obtain the internal temperature, always use the "BODY" mode. It is not designed to measure the temperature of other animals or human temperature.

INTRODUCTION

The Helpmation Infrared Thermometer for Dogs has been developed using the latest infrared technology. This technology allows your dog's temperature to be taken at a distance of up to 3-5cm away from the Inner ear. Precise, instantaneous and touch-free, the Helpmation Infrared Thermometer for Dogs is the most suitable thermometer for no-risk temperature measurement for dogs. This will allow for quicker medical treatment, resulting in a more favorable outcome. This thermometer can be your first diagnostic tool to help keep your dog healthy.

**The normal temperature for dogs is between:
37,2°C to 39,4°C (99,0°F to 102,8°F)**

IF YOUR DOG HAS AN ABNORMAL TEMPERATURE We recommend that you contact your veterinary provider.

Helpmation does not take responsibility for the results or actions of non-medical personnel using the Helpmation Infrared Thermometer for Dogs.

High Temperature Alarm

The factory default alarm is set at 39.7°C(103.4°F). Must be switched the voice guide.

PRECAUTIONS BEFORE USE

The thermometer Helpmation is pre-set at the factory. It is not necessary to calibrate the device when starting.

In order to obtain reliable and stable results, you are advised each time there is a significant change in the ambient temperature due to a change in environment, to allow the thermometer to acclimatize to this ambient temperature for 10 to 15 minutes before using it. It is important to allow 1 minute interval between two measurements.

FUNCTIONS

1. Special design to take your Dog's Temperature from a distance.
2. Reliable and stable measurement, thanks to the advantage Infrared Detection System.
3. Audible alarm if temperature is too high.
4. Memorize the last 32 temperature measurements.
5. Backlit LCD display.
6. Temperature can be displayed in either Celsius or Fahrenheit.
7. Automatic power-off (20 seconds) to save energy.
8. Practical, easy to use.

The thermometer can also be used to measure the temperature of a surfaces or room temperature (by using the Surface Temp or Room function).

USAGE

1. Install batteries (please see Changing the batteries)
2. For the first use or when inserting new batteries wait around 10 minutes for the warm-up of the device.
3. Switch on the unit with the button SCAN.
4. Aim towards the Inner ear, from a distance of 3-5 cm, press again the measuring button SCAN, the temperature is displayed in 1 second.

PRACTICAL CONSIDERATIONS WHEN TAKING A TEMPERATURE

In order to ensure that precise and accurate temperature measurements are obtained, it is essential that each user has received adequate information. Temperature should be take in a neutral context. The dog must not have undertaken vigorous physical activity prior taking the temperature.

A method for measuring the temperature with Helpmation ensure fast and accurate measurement result. The measured temperature can be different according to the method of temperature measurement ie. measured temperature by non-contact thermometer, rectal or ear thermometer could be different.

The measurement the dog's temperature using a different value than the measurement surface and room temperature, so always check before measuring the correct setting to BODY.


HOW TO TAKE YOUR DOG'S TEMPERATURE

Place your dog in a comfortable, draft-free space. Avoid ambient influences such as fans or air conditioning.

The unit is set for reading from the Inner ear of the dog. Depending on the dog, you may choose to use the perianal area, belly, or upper gum (for areas other than the Inner ear, recalibration of the unit may be required – see Key Functions below page 38).

Please make sure to select the BODY mode.

Aim towards the **Inner ear**, keeping the unit at a distance of **3-5 cm** (1.2-2 inches) from the dogs Inner ear, and press the SCAN button.

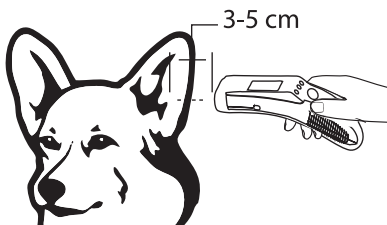
EN

Result will be displayed and/or spoken in one second.
Automatic off within 20 sec.

When measuring the dog can't move
When measuring you can't move the thermometer
Observe the maximum distance 3-5 cm
Measure in the place with the least dog's hairs


POINTS OF INSIGHT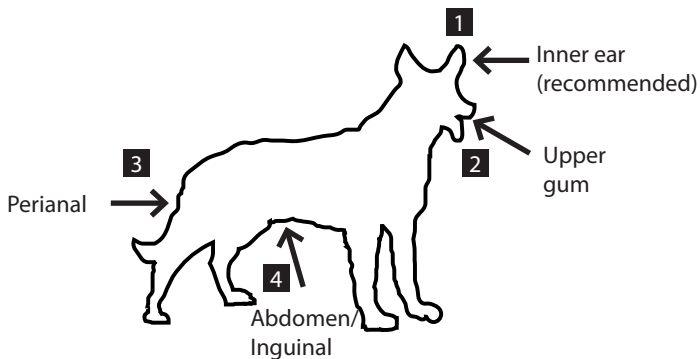

Please note that the lower part of the belly (inguinal area) naturally reads warmer than the area that is closer to the rib cage.

The reliability of the measurement cannot be guaranteed if temperature is measured over any part of the dog's body other than parts mentioned in the section Points of insight.

DESCRIPTION

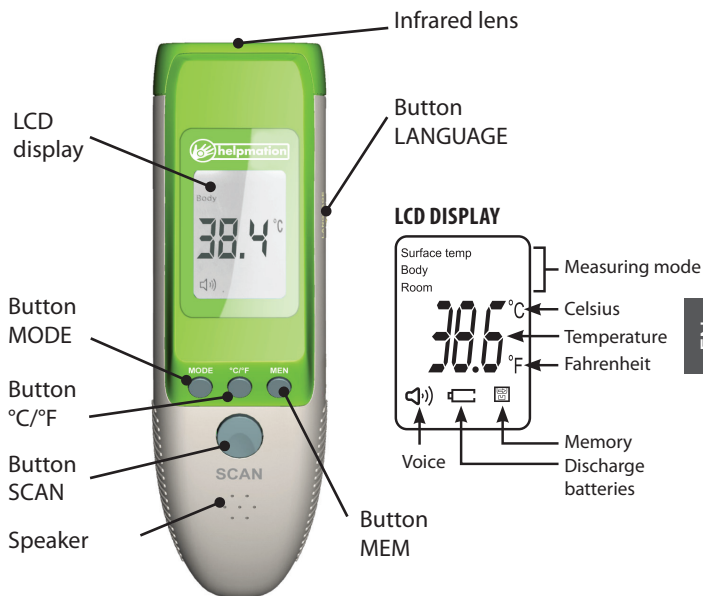SETTING AND FUNCTION

Button SCAN

When the thermometer is off, press the SCAN button once and the thermometer turns on (LCD display lights up). Pressing the SCAN button to measure the desired temperature.

Button MODE

- Press the MODE button and the display mode is displayed **Body** - to measure the dog's temperature
- Press the MODE button again and the display shows **Surface Temp** - for measuring the temperature of baby bottles or baths
- Press the MODE button for the third time and the display shows **Room** - for room temperature

Button °C/°F

Press °C/°F change measurement from degrees Celsius to Fahrenheit and vice versa

Button MEM

Press the MEM button, the display shows the last measured value from newest to oldest (memory thermometer in the last 32 measurements). Pressing MEM displays each temperature thermometer from memory.

Button LANGUAGE

Press LANGUAGE to enable or disable the audio guide thermometer.

CALIBRATION OF THE THERMOMETER

Thermometer factory-set. Depending on various dog's breed, there may be temperature differences. Recalibration of the unit may be required – see Key Functions below.


Press the MODE and °C/°F buttons at the same time for 3 seconds. The screen will then display „F-4“.

Press MODE button to increase 0,1 °C, press °C/°F button to reduce 0,1 °C. Press MEM button to save the setting.

In the cases of seasonal or environmental changes a verification and adjustment should be carried out.


EN

CHANGING THE BATTERIES

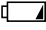When the LCD screen displays , the batteries are used. Do not mix new and old batteries.

Open the lid and change the batteries AAA, taking great care with the correct positioning. (“+” on battery to „+” in battery compartment and „-” on battery to „-” in battery compartment).

A mistake with this could cause damage of the device and compromise the guarantee of your thermometer. Never use rechargeable batteries. Use only batteries for single usage.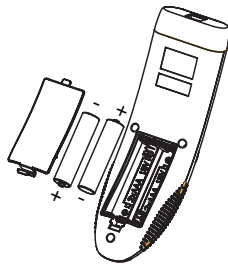

TECHNICAL CHARACTERISTICS

Normal useing condition:

Ambient temperature: 10°C - 40°C (50°F - 104 °F)

Relative humidity: < 85%

Model	RC004
Dimensions	141 x 44 x 52 mm
Weight	110 g (including batteries)
Power	DC3 V (2 baterie typ AAA)
Consumption	< 300 mW
Measuring distance	3-5 cm
Automatic power-off	20 sec.

Measuring range:

In Body mode 32°C - 42,9 °C (89,6 °F - 109,2 °F)

In Surface Temp mode 0 °C - 60 °C (32 °F - 140 °F)

In Room mode 0 °C - 60 °C (32 °F - 140 °F)

Precision from:

36°C ~ 39°C (96,8 °F ~ 102,2 °F) = $\pm 0.2^{\circ}\text{C}$ ($\pm 0,4^{\circ}\text{F}$)

Accuracy: $\pm 0.3^{\circ}\text{C}$ (0.6° F)

Display resolution: 0.1° C (0.1°F)

Contents:

1 pc non-contact thermometer Helpmation

2 pcs AAA batteries

1 pc thermometer protective bag

1 pc instruction manual


The thermometer takes temperature readings below 32 °C or above 42,9 °C (89,6 °F to 109,2 °F) but precision is not guaranteed outside of this range.

TROUBLESHOOTING


If you have problems while using your thermometer, please refer to this guide to help resolve the problem. If the problem persists, please contact our authorised service department.

- **The screen displays temperature higher than 42,9 °C (109,2°F):**
The temperature is in Fahrenheit. Change the measurement to Celsius by pressing °C/°F button.
- **The screen displays temperature lower than 32°C (89,6°F):**
To take the surface temperature, press the MODE button and set to the reading called „Body“. If the device is in Surface Temp mode, the temperature displayed is showing the external temperature of your body, rather than the internal.
- **The measurement values are different**
You must follow up to 3-5 cm distance from the body
You must not move the thermometer during measurement
The dog must not move during measuring the temperature
Measured at the point where are many hair
Measurements must be don in the parts of the body regards manual
The Dog before measurement performed a physical effort
Temperature Measurement dog perform in the place where there is no breeze and not releasing air conditioning

- **The screen displays the message „Hi“:**
When using the thermometer, the message „Hi“ can appear on the screen. In this case, the temperature is above the measurement range selected, for example above 42,9°C (109,2°F) in Body mode.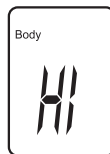

- **The screen displays the message „Lo“:**
When using the thermometer, the message „Lo“ can appear on the screen. In this case, the temperature is above the measurement range selected, for example less than 32 °C (89,6 °F) in Body mode.

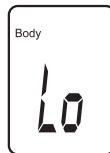This message displays for various reasons:

- measure the dog's temperature in locations as directed
- measurement distance is longer than 3-5 cm
- allow a 1 minute interval between two measurements
- avoid any drafts (e.g. from nasal specs, air conditioning...)

This product is in compliance with the essential requirements and other relevant provisions with EU Directives.

Do not dispose the product in household waste at the end of its useful life. Please, dispose of this device at your applicable collection point for the recycling of electrical & electronic equipments waste.

WARRANTY

Model:

Serial number:

Date of purchase:

Signature and stamp dealer:

The product is guaranteed for 24 months.
The warranty covers defects arising as a result demonstrably defective material, faulty design or bad design.
The warranty does not cover defects caused by transport, normal use of the product (in particular the loss of battery capacity), failure to follow the instructions indicated in the instructions, poor maintenance, mechanical damage, unauthorized intervention in the appliance service. Repairs may only be performed by an authorized service center.

Specifications are based on a condition known at the time of publication.
Technical alterations.

EN

BLAKAR trading s.r.o.
Czech Republic
www.helpmation.com